
# INSTRUCTIONS FOR USE

## LW-02 COLUMN


### COLUMN DESCRIPTION

The LW-02 Column is an immunoadsorption affinity column that selectively removes soluble tumor necrosis factor receptors (sTNF-Rs) from human plasma via an extracorporeal apheresis procedure (Immunopheresis®). The column contains a targeted binding matrix to capture soluble tumor necrosis factor receptor 1 (sTNF-R1), soluble tumor necrosis factor receptor 2 (sTNF-R2), collectively referred to as soluble tumor necrosis factors (sTNF-Rs), housed within a biocompatible polycarbonate cylinder with Luer-lock fluid ports and endcaps. The binding matrix consists of a proprietary single-chain TNF- $\alpha$  (scTNF- $\alpha$ ) polypeptide binding ligand covalently linked to inert agarose beads. Each column has a nominal internal capacity of 33 mL containing a 15 mL bead bed of the binding matrix and 15 mL of storage buffer (total 30 mL solution).

The LW-02 Column is intended to be used in conjunction with the Spectra Optia® Apheresis System (Terumo BCT, Inc.) that uses centrifugal plasma separation techniques that allows for secondary plasma processing and the use of the LW-02 Column in-line within the secondary circuit. The interface between the LW-02 Column and the apheresis system is depicted in Figure 1. An image of the LW-02 Column is provided Figures 2.

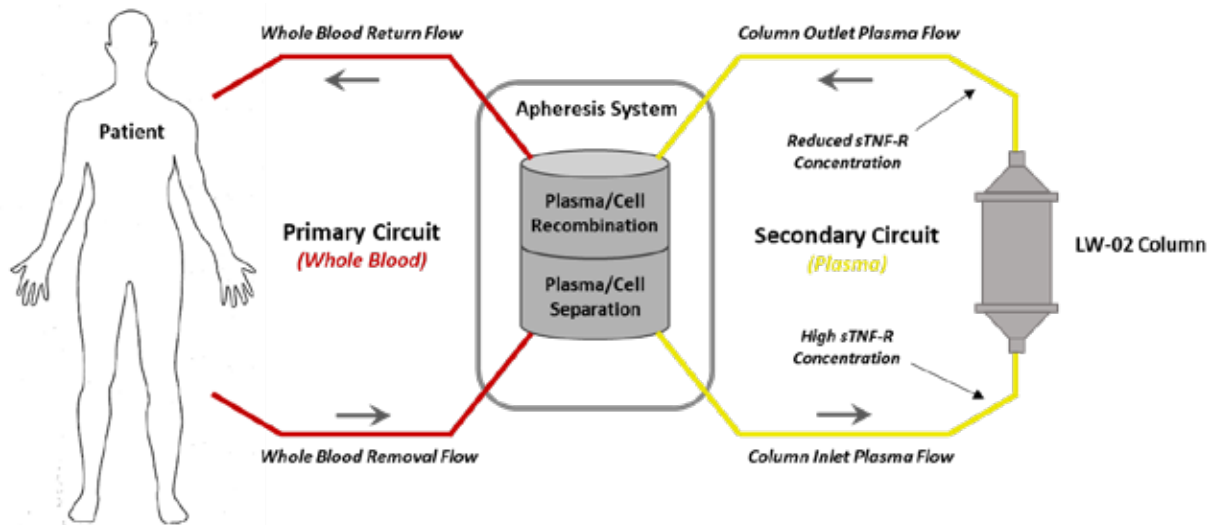**Figure 1 –Interface Between the LW-02 Column and Apheresis System**

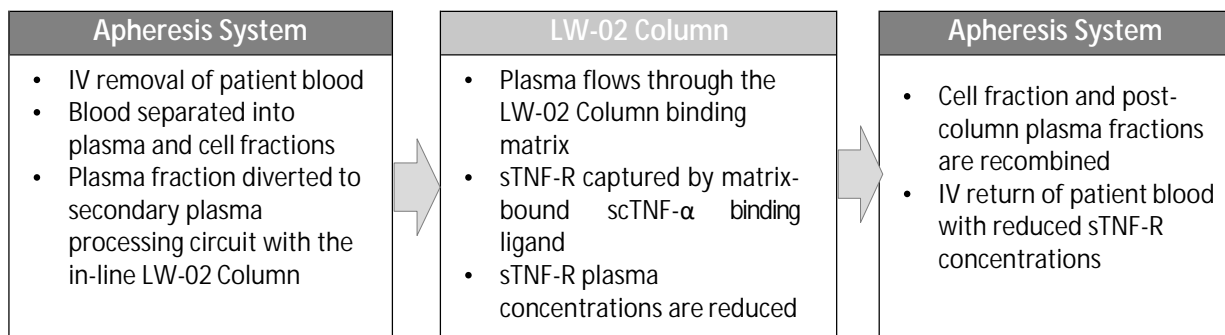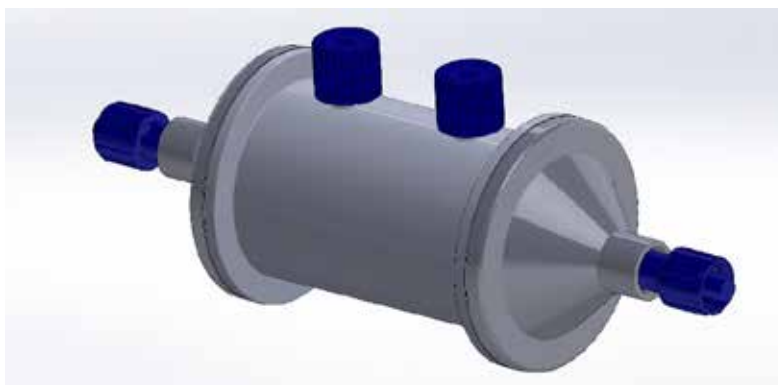
Figure 2 – LW-02 Column

## INTENDED PURPOSE / INDICATIONS FOR USE

The LW-02 Column is indicated for use with the Spectra Optia® Apheresis System (Terumo BCT, Inc.) to remove soluble Tumor Necrosis Factor Receptors (sTNF-Rs) 1 and 2 from the plasma of patients ages 18 and older with advanced, refractory, triple-negative breast cancer. However, the clinical efficacy of the LW-02 Column has not yet been demonstrated. Clinical investigations evaluating the clinical efficacy of the LW-02 Column for this indication are ongoing.

## CONTRAINDICATIONS

Use of the LW-02 Column is contraindicated in patients (that):

- Where extracorporeal circulation therapy is contraindicated. For example, in patients with: severe cardiac insufficiency (New York Heart Association cardiac disease Class III/IV), within 3 months of an acute myocardial infarction, unstable arrhythmias or unstable angina, acute seizure disorder or uncontrolled hypotension or hypertension.
- Currently are receiving angiotensin-converting-enzyme (ACE) inhibitor medications, as extracorporeal blood procedures could cause a potentially life-threatening risk.
- Cannot tolerate therapeutic apheresis procedures or have demonstrated a hypersensitivity associated with either the LW-02 Column or therapeutic apheresis
- Lack adequate vascular access to properly perform apheresis procedures.
- Become pregnant, are pregnant, or are currently breastfeeding.

## WARNINGS

Refer to the Spectra Optia® Apheresis System Operator's Manual for apheresis related cautions for use.

- The LW-02 Column only should be used according to these **Instructions for Use** and under the care and supervision of a physician or other healthcare provider.
- The LW-02 Column is provided sterile. Do not use the LW-02 Column if the packaging is damaged or the pouch seal is broken as the sterility of the column may have been compromised.
- The LW-02 Column is intended to be single-use. Do not reuse or resterilize the column. Reusing the column may pose a serious health risk to the patient and may lead to systemic infection or other complications.
- The LW-02 Column requires priming and flushing prior to use and may be flushed (per physician discretion) at the end of the apheresis procedure to return residual plasma left in the column to the

patient – see **USE AND MAINTENANCE OF THE COLUMN** and see **COLUMN PREPARATION PROCEDURE (PRIMING AND FLUSHING)**.

- Extracorporeal anticoagulation of the apheresis circuit is required in accordance with the apheresis system manufacturer’s operating instructions.
- The LW-02 Column is to be used in conjunction with extracorporeal apheresis systems that use centrifugal plasma separation techniques that allow for secondary plasma processing and the use of the LW-02 Column in-line within the secondary circuit.
- Patients must have adequate vascular access that permits blood flow in excess of 50 mL per minute.
- Vital signs should be monitored; if any clinically meaningful changes are observed, halt treatment and/or provide appropriate supportive care based on physician input.
- Discontinue the procedure in the event of failure or malfunction of the LW-02 Column or apheresis system.
- Should an adverse reaction occur that requires termination of the treatment with the LW-02 Column, treatment should not resume until the physician deems it permissible to do so.
- LW-02 Column Immunopheresis is administered intermittently with an average treatment lasting approximately 3 hours in duration. A single treatment should not exceed 6 hours and the total lifetime contact should not exceed more than 720 hours. Use beyond this timeframe may increase the potential risk of genotoxicity.
  - For example, with an average treatment duration of 3 hours per procedure, a patient that receives 3 treatments per week (resulting in 156 procedures over 1 year) would have a cumulative exposure to scTNF- $\alpha$  of 468 hours over the course of one year (below the recommended maximum 720-hour contact time)."

## **PRECAUTIONS**

- The safe and effective use of the LW-02 Column requires that the user carefully follows both these Instructions for Use and the operating instructions provided by the manufacturer of the apheresis system. Contact Immunicom, Inc., at [product.support@immunicom.com](mailto:product.support@immunicom.com) for further guidance regarding issues with configuration parameters specified herein or with the LW-02 Column.
- Aseptic technique should be followed at all times to ensure patient safety and product quality are maintained.
- Before starting treatment with the LW-02 Column, inspect all intravenous (IV), blood and plasma processing lines to ensure they are not obstructed. Monitor the LW-02 Column and the tubing sets of the primary and secondary circuits for leaks during priming and flushing of the LW-02 Column, and during the entirety of the procedure.
- If Anticoagulant Citrate Dextrose Solution, Solution A (ACD-A) is used as the anticoagulant, proper calcium supplementation is mandatory; ionized calcium levels should be monitored as appropriate, and patients who present signs and symptoms of hypo- or hypercalcemia should be appropriately treated.
- The LW-02 Column apheresis procedure should not be started if the patient exhibits evidence of hypo- or hypercalcemia. Hypo- or hypercalcemia should be corrected prior to initiation of the procedure.
- The LW-02 Column should be used with caution in patients who have underlying or pre-existing conditions that may be exacerbated by the treatment such as:
  - Disorders wherein acute fluid shifts may precipitate congestive heart failure
  - Impaired renal function where expansion or reduction of plasma volume could be harmful

- Clinically significant hypotension or borderline hypotension where a further fall in blood pressure could be harmful
  - Significant vascular disease where a fall in blood pressure could lead to significant post-obstructive physiologic abnormalities
  - Confirmed or suspected intracranial disease where fluid shifts or pressure changes could exacerbate the underlying central nervous system problem
  - Severe anemia (hematocrit < 27%, hemoglobin < 9 g/dL)
  - Systemic infection
  - Pre-existing abnormalities of the coagulation system or a recent history of thrombotic or thromboembolic events that may increase the risk of extracorporeal system clotting or triggering thrombotic or thromboembolic events.
- Based on current clinical safety data, treatment with LW-02 Column should not process more than 2 plasma volumes in one procedure and treatment should be performed no more than three (3) times per week with single treatment not exceeding 6 hours in duration.

## POTENTIAL ADVERSE EFFECTS

Most adverse events reported with apheresis procedures with the LW-02 Column include those associated with any procedure involving extracorporeal blood circulation, plasma processing, or requiring central vascular access. Such procedures carry inherent risks such as infection, blood loss from equipment or tubing leaks, bleeding as a result of improper use of anticoagulants, damage to red blood cells and platelets from the plasma separation centrifugation process, and fluid balance mismanagement leading to hypertension or hypotension.

Potential adverse effects that might be considered anticipated with therapeutic apheresis procedures but unrelated to treatment with the LW-02 Column include:

- |  |                                   |                                    |
|--|-----------------------------------|------------------------------------|
| · Air embolism                               | · Fatigue                         | · Light-headedness                 |
| · Allergic reaction                          | · Fever                           | · Muscle twitching or cramping     |
| · Anemia                                     | · Fluid Imbalance                 | · Nausea and vomiting              |
| · Angina/chest pain                          | · Flushing/blotching              | · Pruritis/itching                 |
| · Anxiety                                    | · Headache                        | · Shortness of breath              |
| · Chills                                     | · Hematoma                        | · Syncope/fainting                 |
| · Coagulopathy                               | · Hemolysis                       | · Thrombocytopenia                 |
| · Device Failure                             | · Hyperkalemia                    | · Thrombosis in patient and device |
| · Diaphoresis                                | · Hyper- or hypocalcemia          | · Urticaria                        |
| · Digit and/or facial paresthesia            | · Hyper- or hypotension           | · Vasovagal reaction               |
| · Discomfort/pain at injection/catheter site | · Hyperventilation                |                                    |
| · Electrical Shock Hazard                    | · Hypogammaglobulinemia           |                                    |
|  | · Hypomagnesemia                  |                                    |
|  | · Inadequate Separation of Fluids |                                    |
|  | · Infection                       |                                    |

Therapeutic apheresis, including the use of extracorporeal columns, have the potential to remove non-target blood constituents and substances (e.g., immunoglobulins, drugs, electrolytes, coagulation factors,

etc.) via nonspecific adsorption or as part of the general apheresis procedure. While the LW-02 Column is designed to be highly specific and selective in binding sTNF-Rs to limit the risk of removing non-target blood constituents and substances, there is potential for unintentional removal of other blood constituents and substances. The long-term effects of such removal have not been fully evaluated. To date, there is no evidence of off-target binding, complement activation, or cytokine release syndrome observed in clinical investigations.

## ADVERSE EVENTS

Although the LW-02 Column is currently indicated for use in advanced, refractory TNBC, Immunicom, Inc. is conducting additional clinical trials for the same column in other cancer types (non-small cell lung cancer, melanoma, and renal cell carcinoma). Adverse event data presented below includes data generated from all ongoing clinical trials, including TNBC. Most of the adverse events observed with the LW-02 Column are common for patients undergoing extracorporeal therapy. Based on accumulated clinical evidence, there is no indication that the LW-02 Column affects the frequency or intensity of these side effects. Additionally, tubing connections require handling increasing the risk of contamination and potential for infection. Further hypersensitivity (and allergic) reactions are possible given that patients are exposed to additional biomaterials.

Data generated from three clinical trials (over 1600 treatments) show treatment with the device across multiple cancers has a consistent and well-tolerated safety profile. In the course of the trials, where LW-02 Column therapy was used alone or in combination with chemotherapy or immunotherapy, there have been 777 treatment emergent adverse events (AEs). The top 10 most frequently reported AEs observed include: anemia (15.6%), neutropenia (5.7%), vomiting (4.1%), thrombocytopenia (4.0%), fatigue (3.6%), chills (3.5%), hypokalemia (3.0%), activated partial thromboplastin time prolongation (2.0%), dyspnoea (2.0%), and hypercalcemia (2.0%). Of the 777 observed AEs, there were 49 (6.3%) were deemed to have a possible, probable or causal relationship to treatment with the LW-02 Column. Of these events, anemia, chills, hypokalemia, pyrexia, and hypocalcemia were the top 5 accounting for 32 of the 49 events (65%). Other events were reported in 2 or fewer cases.

Further, 99 of the 777 AEs also met the criteria to be categorized as serious adverse events (SAEs). Five (5) SAEs - anemia, pleural effusion, dyspnoea, pneumonia and device-related infections accounted for 51 of 99 events (52%). Other SAEs were reported 2 or fewer times and included back pain, diarrhea, fatigue (secondary to anemia), and pancytopenia. Of the 99 SAEs, only 10 (~10%) were deemed to be related to treatment with the LW-02 Column with 4 cases of anemia and one case each of gastric erosive ulcer, COPD exacerbation, pleural effusion, fatigue (secondary to anemia) dyspnoea (associated with respiratory failure) and deep vein thrombosis.

Changes in hematologic parameters (and blood chemistry) are risks with apheresis. Hematological changes, including anemia, neutropenia, and thrombocytopenia also may be causally related to the underlying malignancy and/or to the known toxicity profiles of chemotherapeutic medications. Further, treatment with the LW-02 Column may potentiate the hematologic toxicity of chemotherapeutic medications (e.g., paclitaxel, carboplatin, gemcitabine, irinotecan, etc.), however, this additive effect has not been established.

Treatment with the LW-02 Column appears to be safe and well tolerated and is not known to pose a significant risk for device-related (S)AEs, the possibility of such events, including the possibility of death resulting from treatment with the column cannot be completely eliminated.

The LW-02 Column is being studied when used as monotherapy and in combination with chemo and immunotherapy but has not yet been evaluated in combination with other anticancer treatment

modalities. Additionally, treatment with the LW-02 Column may cause unknown side effects that may be indistinguishable from side effects caused by chemo or immunotherapy or other medications that also affect the immune system.

## INSTRUCTIONS FOR OPERATION

The LW-02 Column is used in conjunction with the Spectra Optia® Apheresis System (Terumo BCT, Inc.). The LW-02 column must be connected to the secondary plasma circuit.

### 1. USE AND MAINTENANCE OF THE COLUMN

The LW-02 Column should be handled with care. Do not use if the sterile packaging is damaged or any caps appear loose or disconnected. Always use aseptic technique when handling the LW-02 Column and when making all tubing connections.

Store the LW-02 Column refrigerated in the range of 2-8 °C (but do not freeze the suspension). Prior to using the column, remove it from cold storage and allow it to reach ambient (room) temperature.

Before connecting the LW-02 Column to the Spectra Optia® Apheresis System's tubing set, invert the column gently several times to enable uniform resuspension of the bead bed within the column. Prime and flush the column prior to use - see **COLUMN PREPARATION PROCEDURE (PRIMING AND FLUSHING)**.

Protect the column from sources of heat and radiation.

The LW-02 Column is a single-use device. Do not resterilize or reuse it.

### 2. SUPPLIES REQUIRED

**Table 1: Materials and Purpose\***

QTY	Material	Purpose
Included:		
1	LW-02 Column	Removal of sTNF-Rs from plasma
Not Included, but required:		
1	Column mounting bracket	Secures the LW-02 Column in the upright position to the apheresis system during the procedure.
2	Waste bag capable of holding at least 1 L of fluid in total	Collects saline used for rinsing/priming the LW-02 Column, that is not intended to be infused into the patient as well as collects the post-treatment rinse. The bag to be disposed of as biohazard waste.
1 L	Normal saline (0.9%) at ambient temperature	Pre-treatment priming (1 L), post-treatment rinse (200 mL), if needed.
1	Stopcock	To control LW-02 Column outlet flow from the waste bag during flushing to the return plasma line during treatment.

*\*Additional materials will be required to properly setup the apheresis system, please refer to the recommendations provided in the manufacturer's operating instructions for the specific apheresis system being used.*

### 3. ANTI-COAGULATION CONSIDERATIONS

The apheresis process involves mechanical separation of plasma and cellular fractions from whole blood. Proper anticoagulation of the apheresis system's extracorporeal circuit ensures that the flow of blood through the circuit is adequate during the procedure and that clotting does not occur in the extracorporeal circuit. The user should follow the anti-coagulation recommendations provided in the manufacturer's operating instructions for the specific apheresis system being used to ensure that an appropriate anticoagulation method is utilized during the procedure. Use of ACD-A anticoagulant solution generally is preferred over other anticoagulants. If ACD-A is used, ionized calcium levels should be monitored, and patients who present signs and symptoms of hypo- and/or hypercalcemia should be appropriately treated. Adequate prophylactic supplementation with calcium (IV or oral) should be provided to the patient either prior to and/or during the procedure.

The attending physician is responsible for prescribing the anticoagulant used, and all personnel involved in performing apheresis procedures should be familiar with the anticoagulant manufacturer's product insert before use.

### 4. COLUMN PREPARATION PROCEDURE (PRIMING AND FLUSHING)

1. Remove the column from cold storage and allow it to come to room temperature prior to use.
2. Inspect and verify that the packaging is intact and that the LW-02 Column is not damaged or leaking. Do not use if the package is damaged or any caps appear loose or disconnected.
3. Prepare the apheresis system to perform a saline prime in accordance with the manufacturer's operating instructions. If prompted, use the parameters defined in Table 2 to configure the apheresis system.
4. Open the plastic pouch containing the LW-02 Column by peeling apart the 2 membranes at the easy-open end.
5. Verify that the two side port caps (located on the side of the column) are securely attached to the column. DO NOT REMOVE the side port caps.
6. The binding matrix bead bed may have separated or settled during storage. Mix the contents of the LW-02 Column by gently inverting it several times, until the binding matrix is homogeneously resuspended in the storage buffer.
7. Using aseptic technique, remove one blue Luer end cap, connect it to the inlet Luer tube line. Invert the device (to ensure flow is from bottom to top) and remove the other blue Luer end cap, connect the stopcock, and connect it to the outlet Luer tubing. The LW-02 Column is bidirectional allowing either end to be used as the plasma fluid inlet.
8. Firmly secure the LW-02 Column to the apheresis system in a vertical orientation (NOT horizontal), ensuring flow is from bottom to top.
9. Save the blue Luer end caps from the column inlet and outlet ports.
10. Connect the LW-02 Column in line with the secondary plasma circuit of the apheresis system in accordance with the manufacturer's operating instructions. However, as standard practice, ***ensure that the plasma will enter the bottom of the column and exit through the top of the column*** (i.e., fluid flowing through the LW- 02 Column always should be directed upward).
11. Prime and flush the LW-02 Column with 1 L of 0.9% normal saline. Ensure that the LW-02 Column and plasma circuit tubing remain free of leaks. Visually verify that all air has been purged from the column and all tubing in the plasma circuit.

### 5. PROCEDURE

1. Prepare the Spectra Optia® Apheresis System in accordance with the manufacturer's operating instructions. Use the parameters defined in **Table 2** to configure LW-02 Column procedure-specific operating parameters.
2. Assure the patient's catheter(s) is/(are) properly connected to the inlet and outlet lines of the apheresis system.
3. Initiate the procedure.
4. Adjust flow rates as needed depending on patient tolerance and the parameters specified in Table 2. Manage anticoagulation (including calcium supplementation if utilized), as prescribed and in accordance with apheresis system operating instructions. The plasma flow rate should not exceed 50 mL/min.
5. Monitor the procedure and observe the patient's general condition, including periodically measuring vital signs according to your institution's standard operating procedure (SOP).
6. At the end of the procedure, follow the apheresis systems instructions and/or physician orders to return the remaining plasma and cellular components to the patient, including the option to rinse the column with 200 mL of 0.9% saline.
7. When the procedure is complete, follow the apheresis system instructions to end the procedure and disconnect the tubing from the patient.

**Table 2: Procedure-Specific Operating Parameters**

Parameter	Description	Value	
Plasma volumes treated	Primary run target. Number of the patient's volumes of plasma to be treated.	2.0	Plasma Volumes
Prime divert volume	Volume of the LW-02 Column, which is also the volume of plasma used to displace the fluid used to prime the LW-02 Column.	25	mL
Plasma pressure alarm limit (Notification)	Limit at which the operator is notified of increasing pressure in the plasma line.	200	mmHg
Plasma pressure alarm limit (Maximum)	Limit at which a plasma line pressure alarm occurs.	250	mmHg
Plasma flow rate limit (Maximum)	Optimal plasma flow rate range is 40-50 mL/min.	50	mL/min
Priming and flushing flow rate	Flow rate of normal saline to flush the column	≤ 100	mL/min

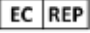
## 6. POST-TREATMENT PROCEDURE

1. When the procedure is complete, follow the apheresis system's instructions to end the procedure and disconnect the patient.
2. Place the LW-02 Column in a biohazard waste bag and discard the column in accordance with the institution's blood products and biohazard waste disposal standard operating procedures (SOPs).

## 7. PATIENT MONITORING

Monitor the patient's vital signs prior, during, and post procedure for approximately 60 minute or per the institutions standard operating procedure (SOP) and physician orders.

## ISO SYMBOLS ON PRODUCT LABELING

	Manufacturer		Do not resterilize
	Authorized representative		Do not use if the product sterile barrier system or its packaging is compromised
	Date of manufacture		Fragile, handle with care
	Use-by date		Protect from heat and radioactive sources
	Batch code		Temperature limit
	Serial number		Do not reuse
	Non-pyrogenic		Consult Instructions for Use
	Sterilized with irradiation		Caution
	Liquid filter with pore size		Importer
	Single sterile barrier system		Manufacturer's catalogue number
	Medical device		
	Immunicom, Inc. 6048 Cornerstone Ct W, Ste. D San Diego, CA 92121, USA		Immunicom Europe sp. z o.o. Ul. Zamknięta 10/1.5 30-554 Kraków, Poland
			

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